


Browser settings for dual-screen image viewing

Enabling dual-screen image viewing requires the corresponding browser permissions.


The settings for Chrome and Edge browsers are as follows:

1、 Chrome browser settings


Click the button on the left side of the address bar, as shown in the image.

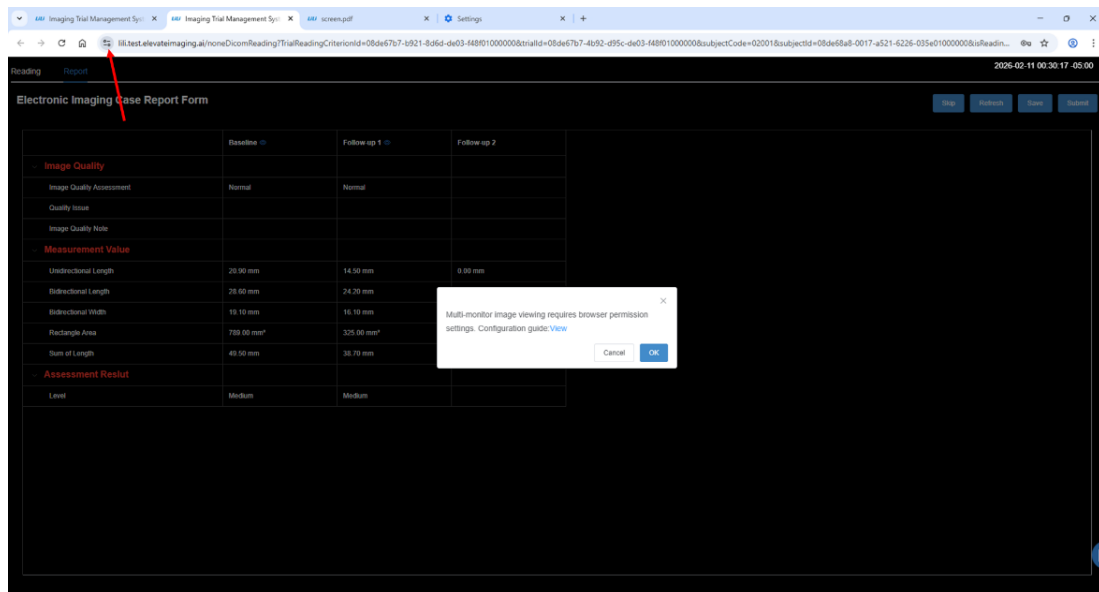


Click on **Site settings**, as shown in the image.

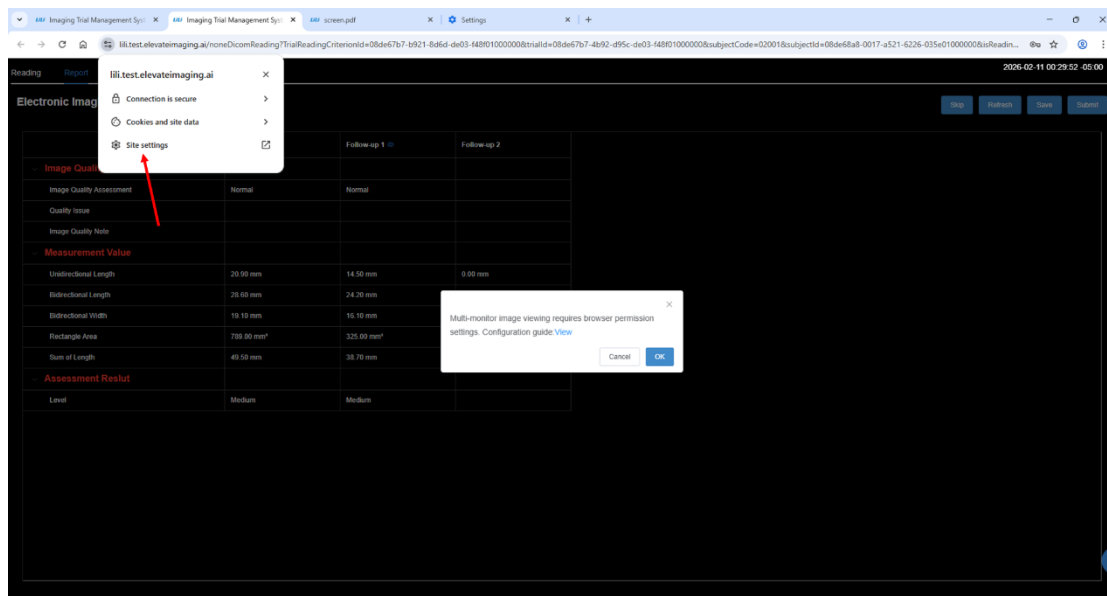

Go to the website settings page, scroll down to find **Window management**, click the drop-down box on the right and select: **Allow**, as shown in the picture.

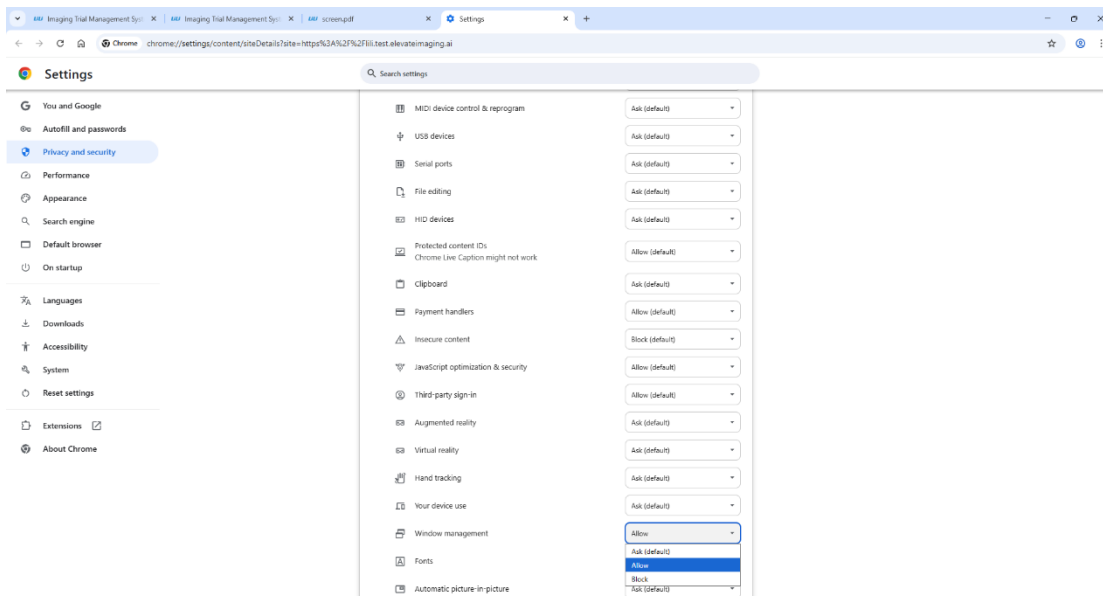


Return to the website, and click "**Reload**". After the page refreshes, click the **View** button again to start view images in dual screens, as shown in the image.

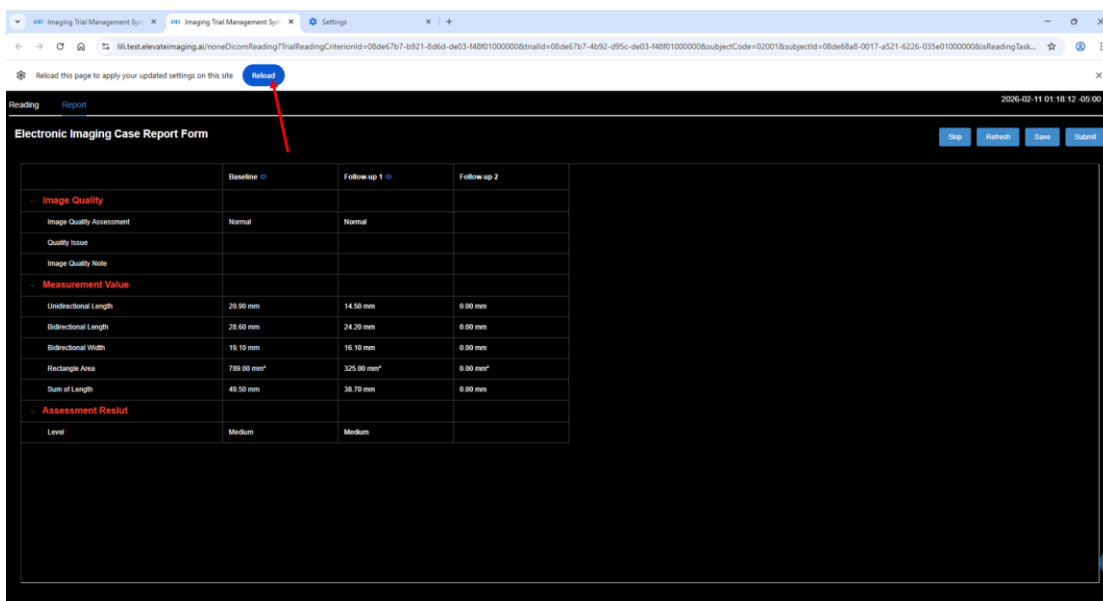


2、 Microsoft Edge browser


Click the button on the left side of the address bar, as shown in the image.

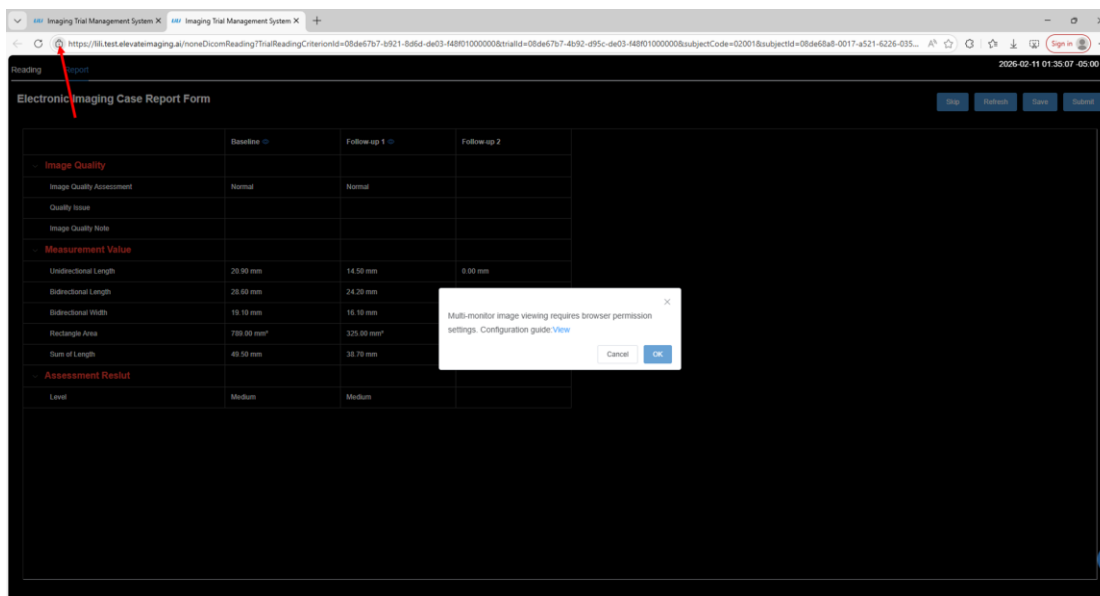

Click on the **Permissions for this site**, as shown in the image.

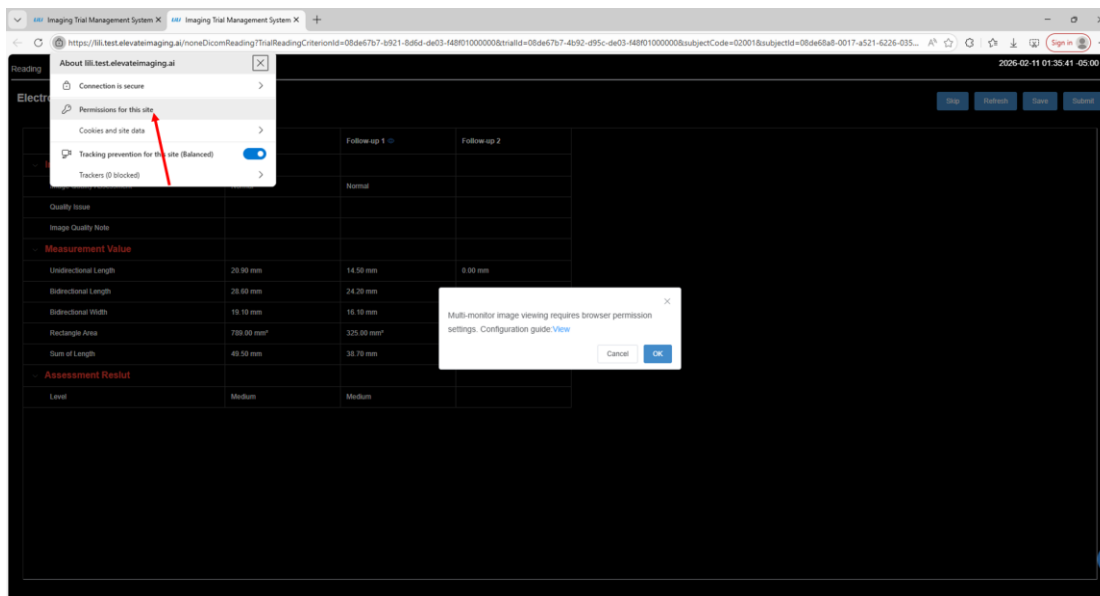

Go to the website permission settings page, scroll down to find **Window management**, click the drop-down box on the right and select: **Allow**, as shown in the figure.

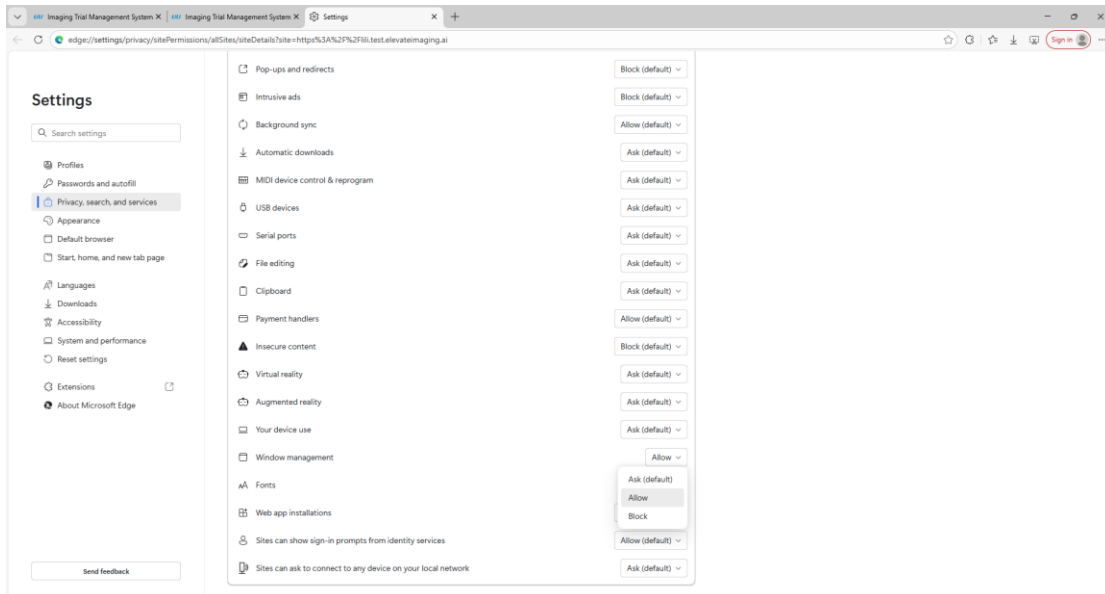

Return to the website, and click **Refresh**. After the page refreshes, click the **View** button again to view images in dual screens, as shown in the image.

